

Warnings:

- It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.
- Do not use in areas with oxygen deficiency.
- Do not use in explosive atmospheres.
- Warning – this product is not flame resistant and must not be used in areas with open flames.
- If the mask is damaged, soaked, dirty, contaminated, mask should be discarded.
- If breathing becomes difficult, dizziness occurs, or other intolerable discomfort – retreat from hazard area and take of the mask and discard it.
- The filtering half mask shall not be used for more than one shift.

Application and Limitations:

These masks shall only be used for protection against COVID-19, they shall not be used for any other purpose like protection against dust, aerosols, etc.

This PPE is designed to protect against the risk from

COVID-19 only

Maintenance

Do not attempt to clean or disinfect mask.
Do not re-use.

How to remove the mask when you are done wearing it:

- Stretch the ear loops and remove the mask only by the ear loops.
- Discard the mask in waste bin.
- Be careful not to touch your eyes, nose or mouth when removing.
- Wash hands immediately after removing mask.

MaskMakers Sweden AB reserves the right to make changes. For the latest version of Instruction for Use, see our website. For more information, Please contact our customers service in Sweden.

+46 18-30 35 30

info@maskmakers.se

Manufacturer

MaskMakers Sweden AB
Förrådsvägen 23
SE-741 30 Knivsta
SWEDEN
www.maskmakers.se

2020-09-21 ©MaskMakers Sweden AB

Important Notice:


MaskMakers Sweden AB does not accept liability of any kind, be it direct or consequential (including, but not limited to, loss of profits, business and/or goodwill) arising from reliance upon any information herein provided by MaskMakers Sweden AB. The user is responsible for determining the suitability of the products for their intended use.

Nothing in this statement will be deemed to exclude or restrict liability of MaskMakers Sweden AB for death or personal injury arising from its negligence.

MM MaskMakers

MM-HM-201

Filtering half mask to protect against COVID-19 only

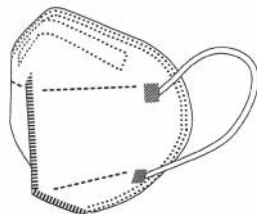


INSTRUCTIONS FOR USE

PPE-R/02.075 version 2



Referring to:

Commision Recommendation (EU) 2020/403
Regulation (EU) 2016/425

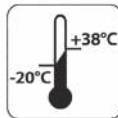



See information supplied by the manufacturer.



Recommended temperature for storage.



End of shelf-life, year/month.



Recommended humidity for storage.

The filtering half mask is manufactured for COVID-19 protection only. As requested by World Health Organization recommendations, for this specific use, the nominal protection factor given by this filtering half mask is the same than the FFP2 nominal protection factor defined in EN 149:2001 + A1:2009. This filtering half mask is not for general use and shall not be used for purposes other than the protection against COVID-19.

Conformity assessment of the PPE


EU type-examination performed by notified body 0200 FORCE Certifications A/S
Park Alle 345, 2605 Brøndby, Denmark

Harmonised standard


PPE-R/02-075 version 2

Before you put your mask on:

- Wash your hands with soap and water or a sanitiser solution (min 60% alcohol).
- Check the mask for holes or other damage.
- Do not use the half mask if it is damaged.
- Discard damaged mask and use a new one.

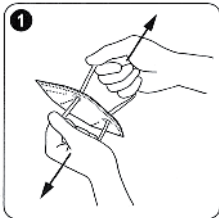

Hold the mask by the ear loops. Open the mask by pulling the ear loops apart.

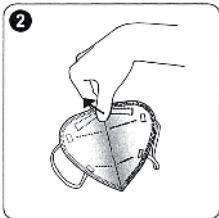


For a better fit straighten the folded nosebridge by pressing at the fold.



Hold the mask in front of your face. Then secure it under your chin and cover your nose.

FITTING INSTRUCTIONS

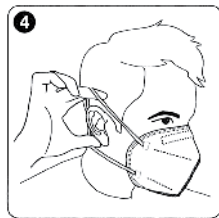

Place the loop around each ear.



Using two hands, starting from the top of the nose, press the metal clip toward the nose, molding the shape of the nose.



Conduct a seal check. Adjust if necessary. The fit is critical to the performance of the product.