

Made in Sweden

Filtering half mask FFP3 NR

- To be Certified by RISE – Research Institutes of Sweden.
- Filtering half mask FFP3 NR against standard EN 149:2001 + A1:2009 to regulation (EU) 2016/425.
- Comfortable, high tech respiratory protection with filtering material in five layers.
- Two-way protection – Protects both user and the environment!
- Protection against virus, bacteria and dangerous particles.
- Next to all input materials are produced within the EU.



MM
MaskMakers
WE PROTECT AND PREVENT
Made in Sweden

Description

Liquid - and splash resistant, high end respiratory protection with high protective degree of protection in challenging environments. The mask is primarily produced for health care professionals but it also complies with standards within industry and helps protect all people exposed to infections/virus and hazardous particles. The mask is designed and produced in Sweden.

MaskMakers respiratory protection, model **MM-HM-501 FFP3 NR**, designed for professional and private use reduces the risk for infections induced by particles and infection/virus found in the close surroundings.

Filtering class

EC Marking FFP2 NR.

Certification

The respiratory protection awaits approval and certification by RISE Research Institutes of Sweden – and fulfils the requirements under EN 149:2001+A1:2009 FFP3 NR to regulation (EU) 2016/425.

Capacity

Filters over 99% of all particles with a size of 0,02 – 2 µm equivalent aerodynamic diameters and with a mass median diameter of 0,6 µm.

Environmental protection

The mask meets the demands for effective protection within health care, public areas and transport.

Technical description

- Five layers of filtering material (amongst others, meltblown and polypropylen) for maximum effect.
- A nosebridge with two-threaded, metal thread, coated with plastic and applied between layers, as distant as possible from the users skin.
- Ear loops made from elastic cord.
- Weight approximately 8 gr.
- Size – One size fit all, foalded approximately 15 x 10 cm.

Allergy

MM-HM-501 FFP3 NR is designed and produced to minimize the risk for skin irritation or allergic reactions.

Instructions for use

The Instructions for use are printed on the Zip-bag and also attached as a printed manual in the package.

Packaging

Standard packaging: 10 pcs filtering masks placed in a handy, resealable ZIP-bag.

Transport carton (standard): 72 units 10-pack, in total 720 pcs filtering masks per transport carton. Weight approximately 12 kg.

Pallet (standard): 16 transport cartons, 1 152 units 10-pack or 11 520 pcs filtering masks. Weight approximately 200 kg. Measurements approximately 80 x 120 x 180 cm.

Manufacturer

MaskMakers Sweden AB. Please, find our contact information below.

Ordering

When ordering, you will receive information of the delivery date. Minimum order quantity: One transport carton containing 72 units 10-pack, in total 720 pcs filtering masks.

Price information


Please, contact your local supplier of personal protective equipment or contact MaskMakers Sweden AB directly.

Respiratory protection


MM-HM-501 FFP3 NR CE

Instructions for use


MM-HM-501 FFP3 NR CE

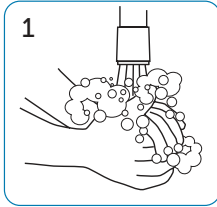


1 Wash your hands with soap and water or a sanitiser solution (min 60% alcohol).

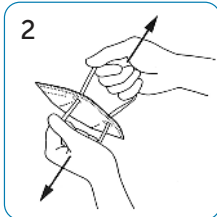


2 Hold the mask by the ear loops. Open the mask by pulling the ear loops apart.

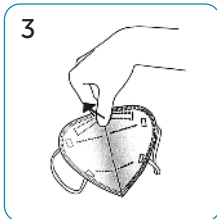

3 For a better fit straighten the folded nose-bridge by pressing at the fold.

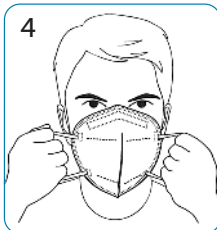


4 Hold the mask in front of your face. Then secure it under your chin and cover your nose.

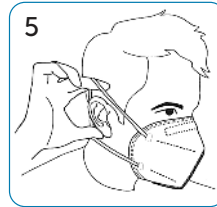


5 Place the loops around the ears.



6 Using two hands, starting from the top of the nose, press the metal clip toward the nose, molding the shape of the nose.

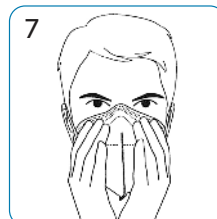

7 Conduct a seal check. Adjust if necessary. The fit is critical to the performance of the product.

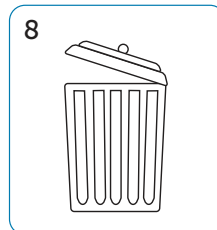

8 Dispose the mask in a waste bin.




✓ **Breathing is easy**

✓ **Filters over 99% of all particles with a equivalent aero dynamic mass median diameter exceeding 0,6 µm.**

✓ **Minimal condensation inside the mask**

✓ **Made in Sweden**

✓ **Protection classification FFP3**

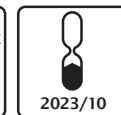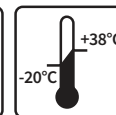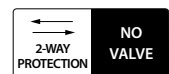

MaskMakers Sweden AB • Förrådsvägen 23, 741 30 Knivsta • SWEDEN
tel: +46-18-30 35 30 • e-mail: info@maskmakers.se • web: www.maskmakers.se
VAT No. SE559244483901 • org.nr. 559244-4839

MM
MaskMakers
WE PROTECT AND PREVENT
Made in Sweden